## What is Shaker Plus?

Shaker Plus is a professional respiratory device for mobilising pulmonary secretions. Shaker Plus has been developed using the latest airflow design technology and requires no additional assembly.

## How does the Shaker Plus work?

Inside the casing of the Shaker is a weighted ball. By breathing into the Shaker Plus, the ball is raised, but then falls under its own weight. This rise and fall action happens extremely quickly and to the user, feels like a vibration. This vibration, combined with the gentle resistance created by breathing against the ball causes a number of positive benefits.

Changes in air pressure mean the smaller airways are encouraged to open, increasing lung usage and efficiency. Mucus is thinned or dislodged by the vibration and increased airflow helps this mucus to move to the large airways where gentle coughing can expel it.

Shaker Plus can be used anywhere, offering simple and convenient relief from excessive mucus and its associated problems. Shaker Plus can be used for chronic conditions such as COPD, asthma, emphysema and acute problems like chesty coughs, flu and bronchitis.

By clearing the airways the chances of developing lung infections and damaging the lungs is reduced. Lung function is also improved, reducing breathlessness and fatigue.

## How to use Shaker Plus

Either sitting or lying down, use your lips to form a seal around the mouthpiece.

Make sure the mouthpiece is in a horizontal position.

Breathe in through the nose and exhale through the mouth.

Position the device so that it generates the maximum vibration in the lungs, which should be easily felt by placing your hand on your chest whilst breathing out.

Gently cough and expel any secretions regularly, or as and when necessary.

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## Precautions

- Shaker Plus is a multi-user device. For hygiene purposes it should be autoclaved before being used by another patient.
- Shaker Plus should not be used by individuals with pneumothorax.
- Individuals with severe cardiovascular disease should only use Shaker Plus on advice from a Doctor.
- Do not make changes to any prescribed medication or prescribed treatment programme without consulting your doctor.
- Adult supervision may be required, when being used by children under the age of 16.
- If you become dizzy while using Shaker Plus, stop the exercise and breathe calmly through the nose until the feeling stops. If dizziness occurs every time you use the Shaker Plus, inhale and exhale without the Shaker Plus every 3rd breath. Should dizziness persist, or any other problems arise whilst using Shaker, please stop using the device immediately and consult your physician.
- Shaker Plus is designed to be used for mobilising pulmonary secretions. No other use is intended or implied. This product is not intended to diagnose, cure or prevent any disease.
- Only devices sold in sealed packaging should be used.
- Store in a cool, dry place, away from direct heat and sunlight.
- This product is sealed to maintain hygiene and cannot be returned if the seal is broken.

## Technical Specifications

Body, cone and lid are made from injected polyamide. The mouthpiece is latex free PVC.

The weighted ball is nickel plated steel.

## Care and Maintenance

Because your Shaker Plus will be exposed to saliva during use, we recommend that your Shaker Plus is cleaned after use. Remove the mouthpiece, unscrew the back case and remove the ball and internal core. Wash all parts in soapy water, rinse with clean water, dry thoroughly and re-assemble. Once a week we recommend that the device is cleansed using approved cleansing tablets.

## Disclaimer

Shaker Plus is not a toy. This product is designed to be used for breathing exercises only. Any other use is not recommended. The material in this manual is for information purposes only. POWERbreath International Ltd makes no representations or warranties with respect to this manual or with respect to the products described herein. No claims are made or implied in the use or results of equipment herein. POWERbreath International Ltd shall not be liable for any damages, direct or incidental, consequential or special, arising out of the use of this material or the products described herein.

Shaker Plus is a patented and design protected product. All rights reserved. Specifications may change without notice due to manufacturers continuous programme of development.

The POWERbreath logo type is a registered trademark of POWERbreath Holdings Ltd.

All POWERbreath product names are trademarks or registered trademarks of POWERbreath Holdings Ltd.

All details are correct at time of going to press.

## Limited 12 month Manufacturers Warranty:

This warranty gives the purchaser specific legal rights. The purchaser may also have other statutory rights. POWERbreath International Ltd hereby warrants to the original purchaser whose name shall be duly registered with it that the product sold by it is free from defects in the material and workmanship.

The obligations of POWERbreath International Ltd under this warranty are limited to the repair and replacement of such parts or parts of the unit as shall be found upon inspection to be defective in material or workmanship. Misuse, abuse or damage, negligence of the user, or failure to maintain or use the unit in accordance with the instructions, or use of unauthorized parts or accessories, or damage caused by service not authorised by POWERbreath International Ltd, or damage resulting from accident, fire, flood, lightning, or other causes not covered under this warranty. During the one year warranty period, the product will be either repaired or replaced (at our option without charge). Warranty does not cover damage or consequential damage caused by service not authorised by POWERbreath International Ltd.

The warranty for professional, commercial or institutional use, is limited to 3 months (90 days) from date of purchase. All other terms remain the same.

The warranties contained herein are expressly in lieu of any other warranties, including implied warranty of merchantability and / or fit for purpose.

To activate your warranty please ensure that you register the purchase of your "Shaker Plus" by visiting [www.powerbreath.com/warranty](http://www.powerbreath.com/warranty).

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Le logo POWERbreath est une marque déposée de POWERbreath Holdings Ltd.