What is Respiron*

Respiron is a flow based lung trainer designed to improve breathing and help restore normal breathing patterns.

Respiron strengthens and exercises the lung muscles by encouraging controlled, slow, deep breathing.

This long, slow, focused breathing has a number of aims and benefits.

- Helps to maintain lung capacity and function after periods of inactivity

- Increases transpulmonary pressure and inspiratory volumes

- Helps post surgical patients restore lung function after heart or lung surgery

- Can be used as part of a Bronchial Hygiene Routine. (Clearance of excess mucus from the lungs)

- Excellent for lowering stress levels

How to use Respiron*

- Connect the mouthpiece and tube to the ball and chamber unit.

- Take the mouthpiece firmly between your lips, holding the unit in an upright position

- Breathe in through the mouthpiece (**do not blow**) to raise the three balls in sequence, keeping them elevated for several seconds. **Please Note:** breathing in must be vigorous and deep, but not quick. The balls should rise in a smooth and uniform movement, rather than suddenly.

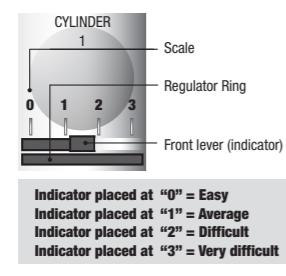
- Remove the mouthpiece at the end of your breath and breathe out normally.

- Take a few normal breaths and repeat as many times as possible, or as many as your health practitioner advises. As a guide, we recommend aiming for 10 breaths, twice a day. If you find this exercise too hard or too easy, the Respiron's exercise intensity and difficulty is adjustable. Please read the next 2 sections for further information.

How to Set The Degree of Intensity of The Exercise Using Respiron*

In order to select the degree of difficulty of the exercise, turn the adjustment ring, using the front lever. The front lever moves according to a scale (0-1-2-3) and works as an indicator showing the degree of difficulty of the exercise.

Once you feel the exercise has become less challenging, the difficulty of the exercise should be increased, by moving the adjustment ring(*) up the scale to the next level. After reaching level 3 the difficulty of the exercise may be further advanced by raising and keeping the balls up for an increasing number of seconds.

ENG

Indicator placed at "0" = Easy
Indicator placed at "1" = Average
Indicator placed at "2" = Difficult
Indicator placed at "3" = Very difficult

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
FRA

How to reduce the exercise intensity with Respiron*

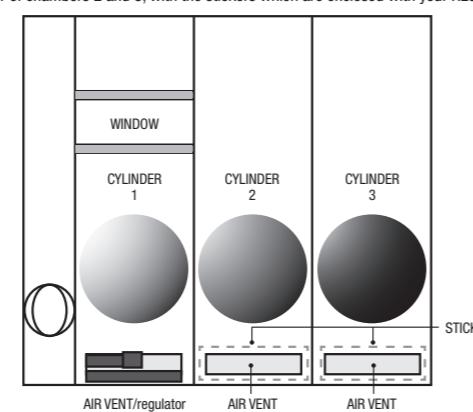Making the exercise easier is especially useful for patients or children who cannot manage to raise the balls at standard level settings. This can be achieved by obstructing the airways located at the base of chamber 3, or of chambers 2 and 3, with the stickers which are enclosed with your RESPIRON.

Please note that each chamber has two openings, one on the front and the other on the back. Start by closing the two openings of chamber 3, using the stickers provided. This will decrease the effort required during the exercise as only the first 2 chambers will fill. If the exercise is still too difficult, close the two openings of chamber 2 as well. The exercise will become even easier, as only the ball of chamber 1 will rise. As the user improves, the stickers may be removed, gradually restoring RESPIRON to its standard levels.

Precautions

- Respiron is a single user device and for hygiene purposes, should not be shared with other users.

- Adult supervision may be required, when being used by children under the age of 16.

- If you become dizzy while using Respiron, stop the exercise and breathe calmly through the nose until the feeling stops. Should dizziness persist, or any other problems arise whilst using Respiron, please stop using the device immediately and consult your physician.

ENG

- Respiron is designed to help strengthen the respiratory muscles. No other use is intended or implied. This product is not intended to diagnose, cure or prevent any disease.

- Only devices sold in sealed packaging should be used.

- Store in a cool, dry place, away from direct heat and sunlight.

This product is sealed to maintain hygiene and cannot be returned if the seal is broken.

Care and Maintenance

Because your Respiron will be exposed to saliva during use, we recommend that your Respiron is cleaned after each use. Remove the mouthpiece and wash all parts in soapy water, rinse with clean water, dry thoroughly and reassemble. Once a week we recommend that the device is cleaned using approved cleansing tablets.

- Keep the equipment inside the packaging while it is not being used.

Inspired air volume (in millilitres) as a function of inspiration time (in seconds) and of the regulator ring position

Inspiration Time	Regulator Ring Position			
	0 (300 mL/s)	1 (400 mL/s)	2 (500 mL/s)	3 (600 mL/s)
3	900	1200	1500	1800
4	1200	1600	2000	2400
5	1500	2000	2500	3000
6	1800	2400	3000	3600
7	2100	2800	3500	4200
8	2400	3200	4000	4800
9	2700	3600	4500	5400
10	3000	4000	5000	6000

If more precise measurements are required, we recommend using specific equipment.

ENG

Disclaimer

Respiron is not a toy. This product is designed to be used for breathing exercises only. Any other use is not recommended.

The material in this manual is for information purposes only. POWERbreath International Ltd makes no representations or warranties with respect to this manual or with respect to the products described herein. No claims are made or implied in POWERbreath International Ltd shall not be liable for any damages, direct or incidental, consequential or special, arising out of the use of this material or the products described herein.

Respiron is a patented and design protected product. All rights reserved. Specifications may change without notice due to manufacturers continuous programme of development.

The POWERbreath logo type is a registered trademark of POWERbreath Holdings Ltd. All POWERbreath product names are trademarks or registered trademarks of POWERbreath Holdings Ltd.

All details are correct at time of going to press.

Limited 12 month manufacturers warranty:

Respiron is produced exclusively for Powerbreath International Ltd for International Distribution. This warranty gives the purchaser the right to repair or replace the product if found to be defective in material or workmanship. The obligations of POWERbreath International Ltd under this warranty are limited to the repair and replacement of such part or parts of the unit as shall be found upon inspection to be defective in material or workmanship. Misuse, abuse or accidents, negligence of the precautions, improper maintenance or commercial use, cracked or broken cases are not covered under this warranty. During the one year warranty period, the product will be either repaired or replaced (at our option without charge). Warranty does not cover damage or consequential damage caused by service not authorised by POWERbreath International Ltd.

The warranties contained herein are expressly in lieu of any other warranties including implied warranty of merchantability and / or fit for purpose.

If mailed in the UK, please send to the following FREEPOST address
(no stamp required). POWERbreath International Ltd
Freepost CV 2407, Southam, Warwickshire CV47 0RD, England, UK

If mailed outside the UK, please send to your local distributor, for more information visit www.powerbreath.com for details.

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This Product is Protected by Patent and Design Copyright ©